



Word/Bartholin Catheter
(Featuring a silicone catheter)

Bioteque is a medical device distributor and can be reached at 800.889.9008 or by fax at 408.526.9399.

Please e-mail us at sales@bioteque.com

Visit our website:
www.bioteque.com

Bioteque America's inflatable Word/Bartholin catheter is made of medical-grade silicone. It is to be used for the treatment of abscesses of the Bartholin gland usually referred to as a Bartholin cyst. The WC-400 catheter set contains the silicone inflatable catheter with a scapel, syringe and needle.

PRODUCT No. WC-400 packaged 6 units/bx

CAUTION

Federal law requires that a physician order this device.

PROCEDURE

Make an incision in the outer wall of the cyst to drain the abscess. After draining the abscess,

insert the deflated silicone catheter through the incision and into the abscess. Using the 3cc syringe, filled with saline solution, carefully inflate the silicone catheter within the cyst cavity by injecting the saline into the catheter. Once inflated, remove the syringe and needle leaving the self-sealing saline inflated Word/Bartholin catheter within the abscess.

After the epithelization of a new orifice is accomplished, the saline solution and catheter are removed from the cyst. After the catheter is removed, the abscess will generally shrink and completely heal leaving the Bartholin gland to function normally.