

Uterine Sound

Visit our website:
www.bioteque.com
 Tel 1-800-889-9008
 Fax 1-408-526-9399



Panpac Medical Corporation
 2~6F, No. 58, Nanxing Rd.,
 Shi-Chih Dist., New Taipei City,
 Taiwan.



STERILE EO




2460

EC REP

Wellkang Ltd
 Suite B, 29 Harley Street
 LONDON, W1G 9QR, U.K.

Not made with natural rubber latex

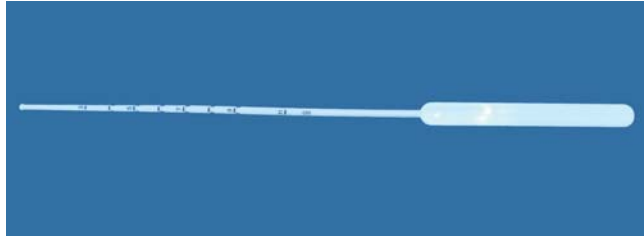

The Uterine Sound is used to gauge the depth and position of the uterine cavity. It is also helpful with the placement of IUDs and the verification of the intrauterine position and length for HSG procedures. The proximal portion of the sound is 3.5 mm in diameter tapering to 2.5 mm at the distal portion with a bulbous tip to reduce the risk of perforation. Malleable to adapt to anatomical variations. It is a sterile, disposable, single-use only device.

DIRECTION FOR USE

The Uterine Sound is a single-use sound. It is made of white, malleable, plastic material. Prior to use, the sound can be curved between the thumb and finger in order to follow the curve of the cervical canal. The graduations are in centimeters and allow for the measurement of the uterine cavity. Dispose of the Uterine Sound in accordance with all applicable Federal, State and local Medical / Hazardous waste practice

PRECAUTIONS

- Do not use if package seal is damaged or broken.
- Minor bleeding, cervical inflammation, and abdominal cramps may occur.

CONTRAINDICATIONS

This disposable Uterine Sound should not be used in the following cases:

- Possible pregnancy
- Infection, including but not limited to Acute Cervicitis and Endometriosis
- Unexplained Uterine Bleeding

WARNINGS

Gently introduce and manipulate the tip of the sound through the cervical canal. Forced entry could create a false tunnel and bleeding.

CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

Do not reuse for avoiding user may be infected.



DO NOT REUSE



ATTENTION



EXPIRY DATE



LOT

BATCH CODE

STERILE EO

STERILE BY ETHYLENE OXIDE



NOT MADE WITH NATURAL RUBBER LATEX